

DBO COMMODE CHAIR CLEANING PROTOCOL

Apply apron and gloves...



STEP 1

Apply brakes, ensure wearing personal protective clothing and access to Multi-Surface Detergent Wipes.



STEP 2

Take out pot cover and place on the floor or on the side This has not been cleaned at this point.



STEP 3

Lift up the arms and clean the front surface of the back rest and seating area. Dispose of wipe.



STEP 4

Remove the cleaned hard seat area and carefully place on the floor



STEP 5

Using a fresh wipe, clean the frame working from top to bottom, finishing with the wheels.



STEP 6

Using a fresh wipe, clean the footrest plates and taking a new wipe then clean both arms thoroughly.



STEP 7

Pick up the clean hard seat and place back on the frame inverted, ensure the clean side is in contact with the clean frame.



STEP 8

Clean the underside of the hard seating area with a fresh wipe, working from the back to the pot area.



STEP 9

Place the clean hard seat back on the frame. Secure the arms and finally clean the lid from the top, around the rim and finish with the underside.



STEP 10

Place cleaned lid onto the hard seat ensuring commode is correctly reassembled.

Remove gloves and apron and cleanse hands.