

RULES OF ULTRASONIC CLEANING

- Before switching on ultrasonic always fill tank with water and use recommended enzymatic detergent at correct dilution.
- Never switch on the heater or ultrasonic unless the tank is full.
- Fill tank to the ridge on inner tank wall or top of the basket.
- Always degass ultrasonic prior to loading instruments in it.
- Change water detergent mixture every day or more often if necessary.
- Always rinse cleaned instruments in clean running water on removal.
- Dry instruments prior to autoclaving with disposable paper towel.
- Always switch ultrasonic off and unplug prior to emptying.
- Rinse out tank to remove soiling and detergent residue, leave dry overnight